


# Williams Medical Supplies

## Sun-WMGB1: User Instructions

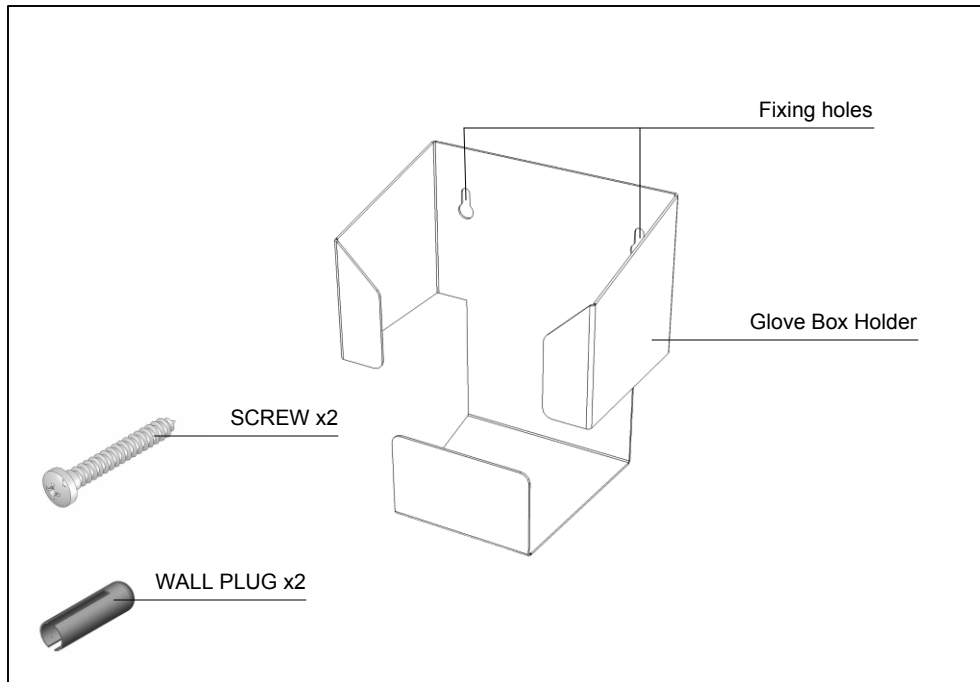

Thank you for purchasing a Williams Medical Supplies Glove Box Holder. Please read and follow these instructions carefully before fixing or use.

Remove the Glove box holder from all the packaging before using the product.

Check the wall is constructed from suitable materials which will support the glove box holder (Please note the plugs and screws supplied are only intended for use in solid walls, typically brick or concrete block). **If in doubt please consult a qualified tradesman or builder.**

Mark out the holes centres using the glove box holder as the template, ensuring it is level. Remove the holder and then drill two holes. Insert the wall plugs supplied and then fix the glove box holder in place using the supplied screws. Check the glove box holder is secure prior to use.

### **Maintenance and Cleaning:**

Periodically (at least once a month) please ensure that the following is checked:

Any bolts, nuts and screws are tight. If any are loose then the product must NOT be used until someone competent has inspected and re-tightened the bolt, nut or screw.

The glove box holder should be cleaned using warm water and a mild soap or diluted disinfectant. Allow to dry before use.

Do not use any abrasive cleaners, bleach or white spirit or allow to become wet.